

GREAT PROCEDURE TIPS FOR IDENTIUM®




TIPS

- > Remove any contaminants prior to taking final impression. (Residue left behind from temp fabrication or injectable hemostatic pastes need to be removed with an alcohol soaked pellet.)
- > Leave used tip on until next use or place original cap back on to avoid material drying out.
- > Do not desiccate the area. Leave the prep area and surrounding teeth glistening. Identium® works best in a wet environment.
- > Polyether single-use syringes (purple) are not compatible with Identium®
- > If using an impression syringe, be sure to thoroughly clean the inside prior to using it with the Identium®.
- > Identium® is very thixotropic and is designed that way on purpose so it will stay where it is placed. The material has flowable properties when it is compressed with the pressure of the tray material.

Note: Products containing sulfur, aluminum chloride, ferric sulfate, inhibit the polymerization of impression material and need to be thoroughly rinsed prior to taking the final impression.

Note: only the green PVS single-use syringes can be used.

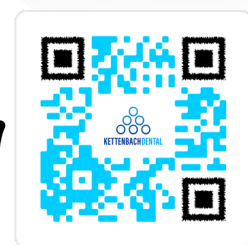

Scan me!

E-mail: custservice@kettenbachusa.com
Call 877-532-2123 www.kettenbach-dental.us