



## FREE Diamonds with Each Case

Receive 2 FREE chamfer prep diamonds with all digital crown & bridge cases **(\$18 Value)**

## Adding Maverick to Your Scanner

### Trios by 3Shape

1. Go to [portal.3shapecommunicate.com/login](http://portal.3shapecommunicate.com/login) in your web browser
2. Connect with Maverick Dental Lab as Lab by searching [impressions@maverickdental.com](mailto:impressions@maverickdental.com)
3. After connecting to us as a Lab, select Maverick Dental Lab when sending files

### Itero

1. Call 800-577-8767
2. Select Option 1
3. Request that Maverick Dental Lab is added to your scanner and identify our lab using our iTero number 8445 or phone number: 866-294-7444
4. After Maverick Dental Lab has been added, restart your scanner
5. After connecting to us as a lab, select Maverick Dental Lab on your scanner when sending files

### Dentsply Primescan & Cerec

1. Create or Login to your CEREC Connect account
2. Select "MY CEREC CONNECT"
3. Type "Maverick Dental Lab" in the box for Company Name
4. Click FIND
5. Check the box to the bottom left for "Maverick Dental Lab"
6. Click ADD to complete the registration
7. When sending cases, select Maverick Dental Lab

### Carestream/DEXIS

1. Select "Find a Lab" option on your scanner
2. Search for either Maverick Dental Lab or [impressions@maverickdental.com](mailto:impressions@maverickdental.com)
3. Add Maverick Dental Lab
4. Select Maverick Dental Lab when submitting scans

### Medit

1. Visit [meditlink.com](http://meditlink.com), login or click on "don't have an account? sign up"
2. Click on "administrator" under whichever category fits your business
3. Fill in your information, and check your email for the verification email / instructions to download the software
4. Once completed, click "Add New Partner" and search for "Maverick Dental Laboratories" to begin sending cases to our lab

### Planmeca®

1. Visit [ddxdental.com](http://ddxdental.com)
2. Sign up as a Dental Practice
3. In the "add lab" section, search Maverick Dental Lab and request account