

Great Procedure Tips for Silginat® Impression Material



TIP#1

Seat vertically. Lean the patient back slightly in the chair and seat the impression slowly and straight up (maxillary) or straight down (mandible).

- > Make sure the tray handle is aligned with the patients midline.
- > Hold the impression tray in the premolar area for stability once seated.



TIP#3



DO NOT

- > Wiggle the tray once in the mouth.
- > Roll the impression from front to back or back to front when seating.
- > Rock tray into place.



DO

- > Seat vertically.



TIP#2

Wait a few seconds after filling the tray to let it firm up a bit. This tends to help with drags. If mixer/dispenser is in another location from the operatory then there is no need to wait.



ADVANTAGES AND BENEFITS

- > Silginat® impressions can be poured immediately following removal from the mouth and also allows for multiple pours.
- > Remains dimensionally stable and does not swell or dry out.
- > Can be sent directly to the lab without pouring up.
- > High flowability – Material flows easily and combines crisp detail and precision for optimal preliminary impressions.
- > Easy removal from mouth, low tear resistance, releases from undercuts.
- > Doesn't need to be measured or mixed.
- > High thixotropy – No unwanted flowing of material into the pharyngeal cavity.
- > Disinfection capable – Impressions can be easily disinfected.



USE

- > Opposing Arches
- > Bleach trays
- > Night guards
- > Sport guards
- > Preliminary impressions
- > Study models
- > Temporary matrix



Want to see
Silginat®
in action?



Scan me

**Buy smart.
Buy direct.**
1-877-532-2123

E-mail: custservice@kettenbachusa.com
Call 877-532-2123 www.kettenbach-dental.us